
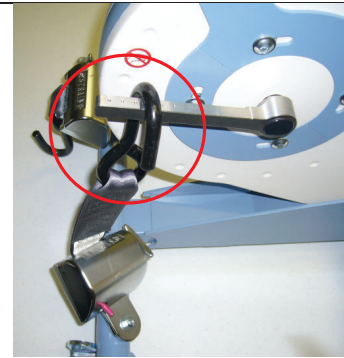




Calibrating the RT300 Ergometer

Leg ergometer calibration


1. Power on the RT300 and wait for the main "GO" page to appear
2. Remove the pedals from the RT300
3. Tap Menu and then Login
4. Enter the calibration code 4102410039 and then tap OK in the bottom right
5. Tap Start Calibration
6. Lock the pedals in place by attaching the wheelchair restraint to the crank

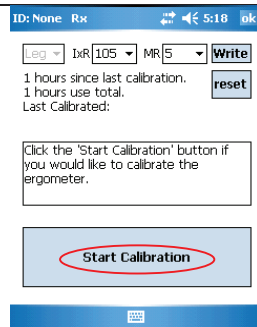

7. Tap "Crank is locked" and the first phase of the calibration process will commence
8. Tap "Click to continue"

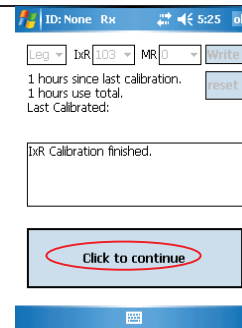
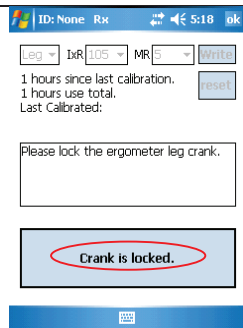


9. Remove the wheelchair restraint from the pedal crank so that it can rotate freely
10. Tap "Crank is unlocked"

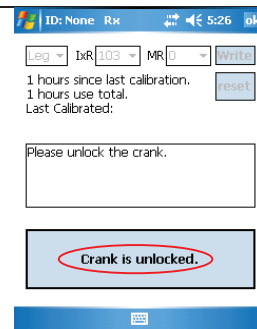

11. Wait for the 2nd phase of calibration process
12. Tap "Click to close dialog"


to complete

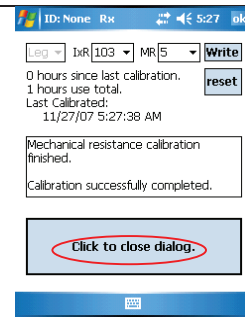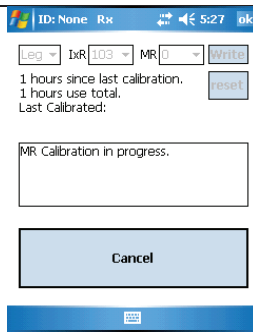



-
13. Refit the pedals. Be sure to tighten the retaining bolt well
-

Arm ergometer calibration


1. Power on the RT300 and wait for the main "GO" page to appear
2. Remove the pedals from the RT300
3. Tap Menu and then Login
4. Enter the calibration code 4102410039 and then tap OK in the bottom right
5. Select Arm and then tap Start Calibration
6. Lock the arm crank in place by turning the arm crank lock knob until it "clicks into place"

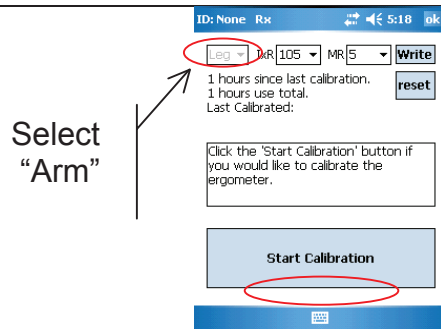

7. Tap "Crank is locked" and the first phase of the calibration process will commence
8. Tap "Click to continue"

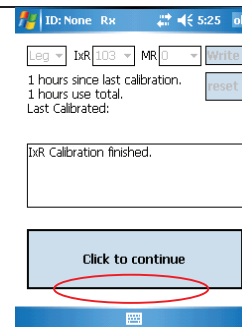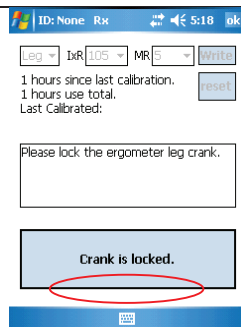

9. Release the arm crank lock knob by pulling it down and turning it 1/2 turn in either direction
10. Tap "Crank is unlocked"

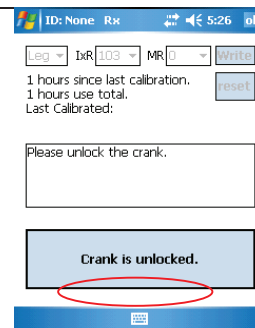

11. Wait for the 2nd phase of calibration process to complete

12. Tap "Click to close dialog"

